

Hydrogen Peroxide Cleaner Disinfectant Wipes

Ready-to-Use, One Step Cleaner Disinfectant



HEALTHCARE™

Ideal for Daily Use Throughout the Facility



Proven Efficacy & Fast Contact Times

- ▶ Hospital Grade Disinfectant Cleaner
- ▶ Kills bacteria, viruses and TB in 1 min or less
- ▶ Stops the spread of infection in your healthcare facilities
- ▶ Effective against organisms responsible for leading HAIs

Easy to Use, Simplifies Compliance

- ▶ One-step, ready-to-use hospital cleaner disinfectant wipe
- ▶ No mixing, measuring, or rinsing required
- ▶ Delivers consistent amount of solution concentration every time
- ▶ Larger wipe covers more surface area
- ▶ Labour saving no-rinse formula*
- ▶ Stays wet for entire contact time

Helps to protect your patients and staff

- ▶ Does not contain benzyl alcohol
- ▶ No fragrance added
- ▶ No harsh chemical fumes, odour, build-up or residue
- ▶ Does not contain bleach
- ▶ Non-flammable



DIN 02406225

Number of pathogens killed in 30 seconds	35
Number of pathogens killed in 1 minute	13
Total kill claims	56

Now kills TB in 1 minute

*rinsing required on food contact surfaces.

Clorox Healthcare™
Hydrogen Peroxide Cleaner Disinfectant Wipes


		CONTACT TIME	
PATHOGENS		DIN 02406225	
BACTERIA	<i>Acinetobacter baumannii</i> (A. baumannii)	30 SEC	
	<i>Campylobacter jejuni</i> (C. jejuni)	30 SEC	
	<i>Enterobacter aerogenes</i>	30 SEC	
	Multidrug Resistant <i>Enterococcus faecium</i>	30 SEC	
	Vancomycin Resistant <i>Enterococcus faecalis</i> (VRE)	30 SEC	
	<i>Escherichia coli</i> (E. coli)	30 SEC	
	<i>Escherichia coli</i> O157:H7 (E. coli)	30 SEC	
	<i>Escherichia coli</i> New Delhi Metallo-Beta Lactamase-1 (NDM-1)	30 SEC	
	<i>Escherichia coli</i> with extended beta-lactamase resistance	30 SEC	
	<i>Klebsiella oxytoca</i>	30 SEC	
	Carbapenem Resistant <i>Klebsiella pneumoniae</i> (CRKP)	30 SEC	
	<i>Klebsiella pneumoniae</i> with extended beta-lactamase resistance	30 SEC	
	<i>Klebsiella pneumoniae</i> New Delhi Metallo-Beta Lactamase-1 (NDM-1)	30 SEC	
	Multidrug Resistant <i>Klebsiella pneumoniae</i> (MDR K. pneumoniae)	30 SEC	
	<i>Pseudomonas aeruginosa</i> (P. aeruginosa)	30 SEC	
	<i>Salmonella enterica</i> (S. enterica)	30 SEC	
	<i>Stenotrophomonas maltophilia</i> (S. maltophilia)	30 SEC	
	Penicillin-Resistant <i>Streptococcus pneumoniae</i> (PRSP)	30 SEC	
	<i>Streptococcus pyogenes</i> (S. pyogenes)	30 SEC	
	Community Acquired Methicillin Resistant <i>Staphylococcus aureus</i> - CA-MRSA	45 SEC	
	Linezolid Resistant <i>Staphylococcus aureus</i>	45 SEC	
	Methicillin Resistant <i>Staphylococcus aureus</i> - MRSA Hospital Acquired	30 SEC	
	Methicillin Resistant <i>Staphylococcus aureus</i> - MRSA	30 SEC	
	<i>Staphylococcus aureus</i>	1 MIN	
	Coagulase Negative <i>Staphylococcus epidermidis</i> (CoNS)	45 SEC	
	<i>Bordetella bronchiseptica</i>	1 MIN	
	MYCOBACTERIA	<i>Mycobacterium terrae</i> (TB)	1 MIN

Clorox Healthcare™
Hydrogen Peroxide Cleaner Disinfectant Wipes

		CONTACT TIME
PATHOGENS		DIN 02406225
VIRUSES	Avian Influenza A (H5N1) virus	30 SEC
	Cytomegalovirus	30 SEC
	Duck Hepatitis B virus (surrogate for human Hepatitis B virus)	30 SEC
	Hepatitis C virus (Bovine Viral Diarrhea virus as surrogate)	30 SEC
	Herpes simplex virus type 1	30 SEC
	Herpes simplex virus type 2	30 SEC
	Human Coronavirus	30 SEC
	Human Immunodeficiency Virus type 1 (HIV-1)	30 SEC
	Influenza A (H3N2) virus	30 SEC
	Influenza A virus ([2009]H1N1)	30 SEC
	Influenza B virus	30 SEC
	Respiratory syncytial virus (RSV)	30 SEC
	Canine Distemper virus	30 SEC
	Feline Calicivirus (Surrogate for Human Norovirus)	1 MIN
	Rhinovirus (Type 37)	1 MIN
	Rotavirus	1 MIN
	Canine Influenza (H3N2) Virus	1 MIN
	Canine Influenza (H3N8) Virus	1 MIN
	Feline Infectious Peritonitis virus	1 MIN
	Feline Rhinotracheitis virus	1 MIN
	Pseudorabies virus	1 MIN
	Human Adenovirus type 5 (BSV)	1 MIN
	SARS-CoV-2 (virus that causes COVID-19)	30 SEC
MOLD, FUNGI:	Candida albicans	3 MIN
	Trichophyton interdigitale (formerly Trichophyton mentagrophytes)	3 MIN
	Microsporum canis	1 MIN
	Candida auris	3 MIN
	Aspergillus niger	4 MIN

Formulated for better surface compatibility and less residue than ever before

Visibly Less Residue vs. Other Disinfectants

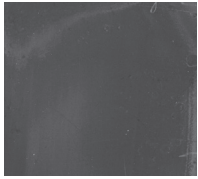


Clorox Healthcare™ Hydrogen Peroxide Cleaner Disinfectant Wipes



Major Hydrogen Peroxide Wipes Brand


(surface: black ceramic)

Faster Cleaning Power than Quat Alcohol

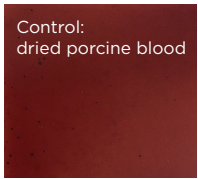

Control: dried porcine blood

(surface: white ceramic)

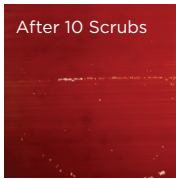

After 10 Scrubs

Leading Quat Alcohol Wipes Brand



After 30 Scrubs



After 10 Scrubs

Clorox Healthcare™ Hydrogen Peroxide Cleaner Disinfectant Wipes



After 30 Scrubs

Suitable on Surfaces Even After Long-Term Use



Control, No Wiping



After 10 Wipes



After 90 Wipes



After 180 Wipes

Clorox Healthcare™ Hydrogen Peroxide Cleaner Disinfectant Wipes (surface: stainless steel)

70% of IPs/EVS believe residue negatively impacts their audit scores.¹

Designed for Use on Common Medical Surfaces[†] Including:

- ▶ Bed rails
- ▶ Bedside tables
- ▶ Bed frames
- ▶ Blood pressure cuffs^{††}
- ▶ Blood pressure monitors
- ▶ Diagnostic equipment
- ▶ Exam tables
- ▶ Handles
- ▶ IV poles
- ▶ Keyboards
- ▶ Laptops
- ▶ Operating room tables and lights
- ▶ Tray tables
- ▶ Wheelchairs
- ▶ X-ray equipment

Ideal for Use in High Turnover Areas Throughout the Facility

- ▶ Ambulatory Surgical Centers
- ▶ Dialysis Clinics
- ▶ Emergency Rooms
- ▶ Intensive Care Units
- ▶ Operating Rooms
- ▶ Patient Rooms
- ▶ Radiology or X-Ray Rooms

¹ Online survey of 208 healthcare disinfectant decision makers, April 2017.

[†]Use as directed on hard, nonporous surfaces

^{††}hard, nonporous surfaces of

Hydrogen Peroxide Cleaner Disinfectant Wipes

FOR HARD NONPOROUS SURFACES



Multipurpose Wipes (17.1 cm x 22.8 cm)

Designed for: Nurses and Environmental Services

Recommended for: everyday cleaning and disinfecting, daily patient room cleaning, high-touch surfaces.

Item No. 01456, 95 ct. canister, 6/case

DIN 02406225



Terminal Wipes (30.4 cm x 27.9 cm)

Designed for: Environmental Services

Recommended for: Terminal cleaning and disinfecting of hard surfaces.

Item No. 01458, 185 ct. bucket, 2/case

Refill Item No. 01459, 185 ct. refill, 2/case

DIN 02406225

